
nedis

In-ear Wireless TV Headphones

light, comfortable stethoscope-style neck band

HPRF021BK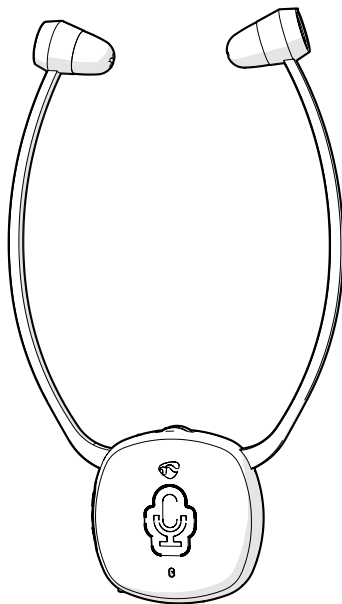

ned.is/hprf021bk

EN	Quick start guide	4
DE	Kurzanleitung	6
FR	Guide de démarrage rapide	8
NL	Snelstartgids	11
IT	Guida rapida all'avvio	13
ES	Guía de inicio rápido	16
PT	Guia de iniciação rápida	18
SV	Snabbstartsguide	20
FI	Pika-aloitusopas	23
NO	Hurtigguide	25
DA	Vejledning til hurtig start	27
HU	Gyors beüzemelési útmutató	29
PL	Przewodnik Szybki start	32
EL	Οδηγός γρήγορης εκκίνησης	34
SK	Rýchly návod	37
CS	Rychlý návod	39
RO	Ghid rapid de inițiere	41

In-ear Wireless TV Headphones HPRF021BK

For more information see the extended manual online:
ned.is/hprf021bk

Intended use

The Nedis HPRF021BK is a wireless audio amplification system designed for people with hearing difficulties. It connects with different kinds of audio systems, such as a TV set, smartphone, DVD player, CD player, Hi-Fi system etc. The transmitter also functions as the charger for the headphones. Multiple receivers can be paired with one transmitter. This product can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the product in a safe way and understand the hazards involved. Children shall not play with the product. Cleaning and user maintenance shall not be done by children without supervision. Any modification of the product may have consequences for safety, warranty and proper functioning.

Specifications

Product	In-ear Wireless TV Headphones
Article number	HPRF021BK
RF frequency range	2406 - 2472 MHz
Max. transmit power	10 dBm
Operating range	Up to 25 m
Maximum Playback Time	Up to 7 hours
Battery capacity	380 mAh
Maximum Charging time	Up to 3 hours
Automatic switch off	After 5 minutes of inactivity
Balance control	Yes
Volume control	Yes
Microphone amplification	Yes
Hearing impaired support	Yes
Power input	5.0 V DC / 550 mA
Connections	Optical Toslink input AUX input
Dimensions (w x h x d) Headphones	113 x 235 x 20 mm

Dimensions (w x h x d) Transmitter	117 x 117 x 38 mm
Weight	160 g

Main parts (image A)

- | | |
|---------------------------|---|
| 1 Headphones | 12 3.5 mm audio cable (male – male) |
| 2 Volume wheel | 13 RCA adapter cable (3.5 mm female – 2 x 2.5 RCA male) |
| 3 Indicator LED | 14 Optical audio cable (Toslink) |
| 4 Mute button | 15 Power adapter |
| 5 On / off switch | 16 Earbuds |
| 6 Microphone | 17 Sound balance switch |
| 7 Charging contacts | 18 Power input |
| 8 Transmitter | 19 Optical audio input |
| 9 Charging pins | 20 Unit switch |
| 10 Power indicator LED | 21 Audio port (3.5 mm) |
| 11 Charging indicator LED | |

Safety instructions

To prevent possible hearing damage, do not listen at high volume levels for long periods.

⚠ WARNING

- Ensure you have fully read and understood the instructions in this document before you install or use the product. Keep the packaging and this document for future reference.
- Only use the product as described in this document.
- Do not use the product if a part is damaged or defective. Replace a damaged or defective product immediately.
- Do not drop the product and avoid bumping.
- This product may only be serviced by a qualified technician for maintenance to reduce the risk of electric shock.
- Do not expose the product to water or moisture.
- The product is delivered with a partially charged battery. For optimal battery life, fully charge the battery before first use.
- Do not dismantle, open or shred secondary cells or batteries.
- Do not expose cells or batteries to heat or fire. Avoid storage in direct sunlight.
- Do not subject cells or batteries to mechanical shock.
- In the event of a cell leaking, do not allow the liquid to come in contact with the skin or eyes. If contact has been made, wash the affected area with copious amounts of water and seek medical advice.
- To prevent possible hearing damage, do not listen at high volume levels for long periods.

- Using headphones with both ears covered while participating in traffic is discouraged and may be illegal in some areas.
- Some wireless products may interfere with implantable medical devices and other medical equipment, such as pacemakers, cochlear implants and hearing aids. Consult the manufacturer of your medical equipment for more information.
- Do not use the product at locations where the use of wireless devices is prohibited due to potential interference with other electronic devices, which may cause safety hazards.
- This product is equipped with a non-removable internal battery. Do not attempt to replace the battery yourself.
- Charge the battery for at least 3 hours when you use it for the first time.
- Do not use any charger other than that specifically provided for use with the product.
- The charger must only be plugged into an earthed socket-outlet.
- During charging, the battery must be placed in a well-ventilated area.
- Do not leave a battery on prolonged charge when not in use.
- Only power the product with the voltage corresponding to the markings on the product.
- Children of less than 8 years should be kept away.
- Cleaning and user maintenance shall not be done by children without supervision.
- This product is not a toy. Never allow children or pets to play with this product.

Charging the headphones

A1 Charge the battery for at least 3 hours when you use it for the first time.

1. Slide the on / off switch **A5** to the *OFF* position to switch off the product.
2. Place the headphones **A1** with the charging contacts **A7** onto the charging pins **A9**.

The charging indicator LED **A11** lights up red to indicate that the headphones are charging, and turns green when the battery is fully charged.

Connecting the transmitter

1. Plug the power adapter **A15** into the power input **A13**.
2. Plug the other end of **A15** into a power outlet.
3. Choose one of the three options to connect the transmitter.

3.5 mm audio cable

- a. Plug the audio cable **A12** into the audio port **A21**.
- b. Plug the other end of **A12** into your audio source device (e.g. TV or Hi-Fi system).

3.5 mm audio cable + RCA adapter cable

- a. Plug **A12** into **A21**.
- b. Plug the other end of **A12** into the RCA adapter cable (female) **A13**.
- c. Plug the other end of **A13** into your audio source device (e.g. TV or Hi-Fi system).

Optical audio cable

- a. Plug the optical audio cable **A14** into the optical audio input **A19**.
 - b. Plug the other end of **A14** into your audio source device (e.g. TV or Hi-Fi system).
4. Slide **A5** to the *ON* position to switch on the product.
 - i** If there is no audio signal from the audio source, or if the signal is too weak for proper reception, the transmitter turns off automatically after about 5 minutes. As soon as the audio signal returns, the transmitter turns back on automatically.

Pairing an additional headset

You can listen to the same audio source device with more people by connecting multiple headphones to a single transmitter.

Follow these instructions to pair additional headphones to **A3**.

1. Unplug **A15** from the power outlet.
2. Slide **A5** to the *ON* position to switch on the additional headphones.
3. Press and hold the mute button **A4** for 5 seconds.

The indicator LED **A3** blinks quickly to indicate pairing mode is active.

4. Plug **A15** into the power outlet.

A3 lights up continuously to indicate pairing is successful.

Using the product

1. Switch on the audio source that is connected to the transmitter. The power indicator LED **A10** lights up green.
2. Slide **A5** to the *ON* position to switch on the product.

A3 lights up blue.

- i** Rotate the volume wheel **A2** to adjust the volume to the desired sound level.
- i** Rotate the sound balance switch **A17** to adjust the volume balance between the left and the right earbud.

Switching between modes

Follow these instructions to switch between listening mode and conversation mode without taking off the headphones.

1. Press the mute button **A4** to switch to ambient sound mode. Sounds and voices picked up by the microphone are amplified through the headphones.

2. Press **A** **4** again to switch back to your audio source device.
- i A3** starts blinking when the product has around 20 minutes of battery life left.

Declaration of Conformity

We, NEDIS, as manufacturer, declare that product HPRF021BK from the brand Nedis, produced in China was tested according to all relevant CE standards / regulations and passed all tests.

The complete Declaration of Conformity (+ safety datasheet if applicable) can be found and downloaded via <http://webshop.nedis.com>

For other compliant-related information, please contact customer service desk for support:

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NEDIS B.V., De Tweeling 28,